

QUICKEST AND EASIEST URINE COLLECTION DEVICE FOR TRANSPORTATION AND DIRECT USE ON ALIFAX AUTOMATED INSTRUMENTS



Code SI 0903.900

PENOK is a new patented collection device conceived to simplify the urine sample collection that can be performed directly by the patient.

A simple press on the tube allows the sample aspiration and after the removal of the needle the so filled tube is ready to be sent to the laboratory and loaded onto Alifax instruments.

HOW TO USE PENOK



1. Squeeze PENOK and dip the top into the sample. Gently release the tube



2. Uncap lightly pressing the red beak



3. Close PENOK with the white cap



PENOK can be loaded directly into **Alfred 60^{AST}** or Sidecar sample rack for a fully automated analysis



Optional use of **PENOK** for rapid chemical analysis with urine strip

Advantages

- Easy to use and carry
- Economic
- Full sample traceability by unique and unrepeatable barcode
- Direct loading into Alfred60^{AST} and Sidecar rack with no need to open the tube
- Reduced contamination, hands-on-time and exposure
- Compatible with urine dipstick
- One single tube for multiple tests
- CE marked and patent pending